


## 1. Accessing Uniview (via SanCare)


Standard workflow at the San

- Uniview is launched directly from SanCare results or SanApps Specialist Clinical Applications menu
- The correct patient context is passed automatically (please confirm the patient ID)
- Images open without any manual searching

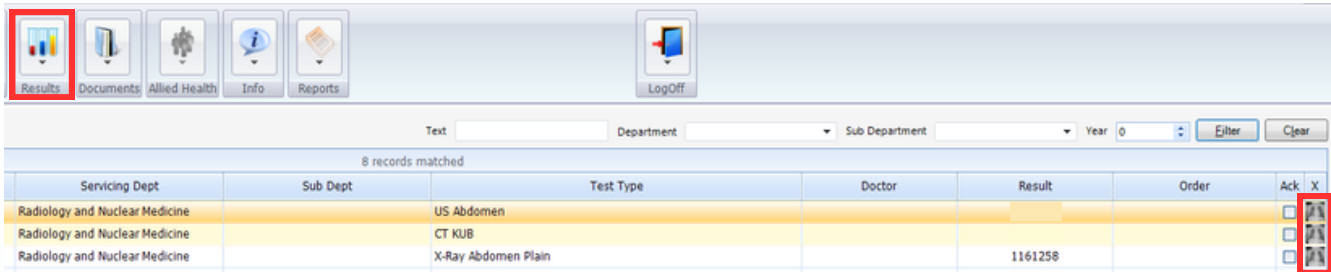


## 2. Patient & Exam Navigation

- Patient demographics appear at the top of the page
- Examination list appears on the left hand side of the page
- Examinations are automatically sorted by date
- To view reports click on documents on the left hand side menu
- Quickly find relevant exams using:
  - Filter examinations box – type to narrow results
  - Filter button – filter by Modality

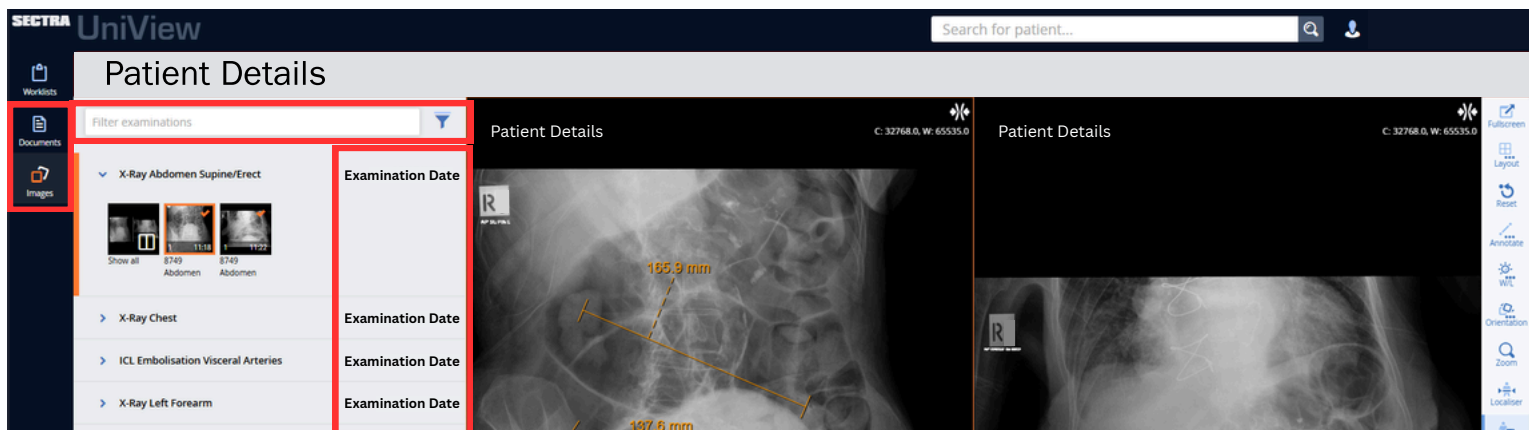


## 3. Opening & Comparing Images

- Open images by clicking on the image thumbnail
- Compare studies:
  - Drag and drop prior exam series into the image window
- Orange tick:
  - Indicates which series are currently displayed
- Exam date is shown in the bottom-left of each image tile

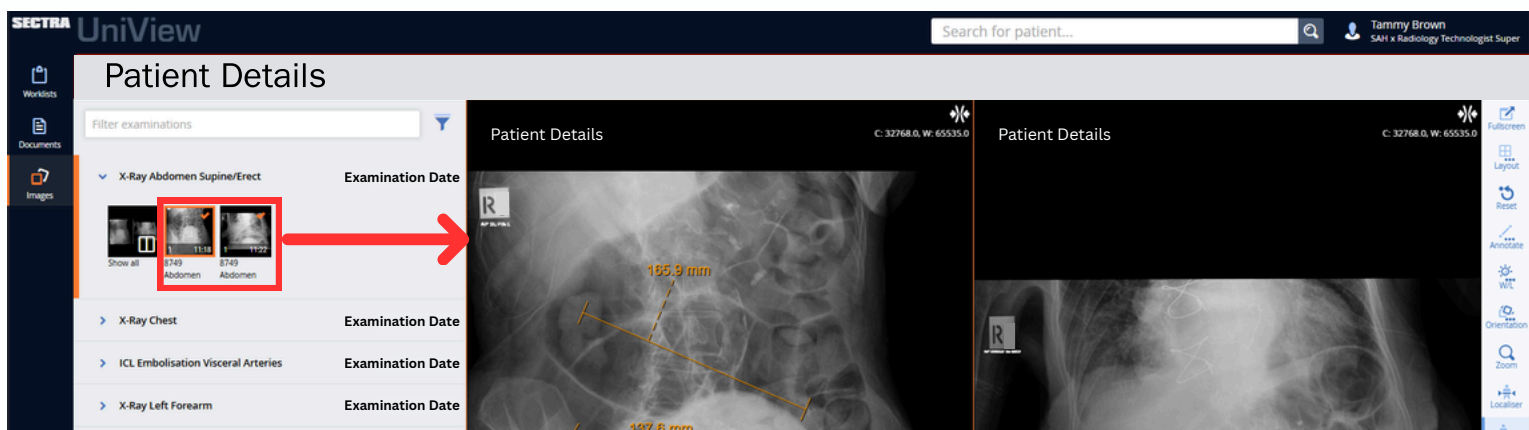

#### 4. Image Layout Controls

- Select the Layout icon (toolbar) to change screen arrangement
- Drag series into specific tiles
- Double-click an image:
  - Enlarges to full screen
  - Double-click again to return to the previous layout

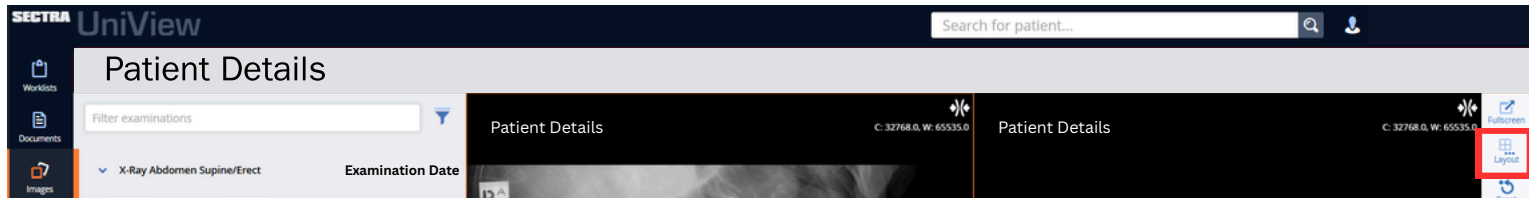

#### 5. Basic Image Controls (Most Used)

Image toolbar is located on the right side


Tip:

- The active image tile is highlighted with an orange border
- For all other shortcuts click on the Shortcuts button at the bottom of the toolbar on the right hand side.

Action	How to Perform
SCROLL SLICES	Mouse wheel OR Left + Right mouse click together
CINE (PLAY STACK)	Click Play ► icon
WINDOW / LEVEL	Click and hold centre button and drag (up/down & left/right)

#### 6. Measurements

- Click on Annotate and select a measurement tool from the right hand toolbar
- Click to place measurement points

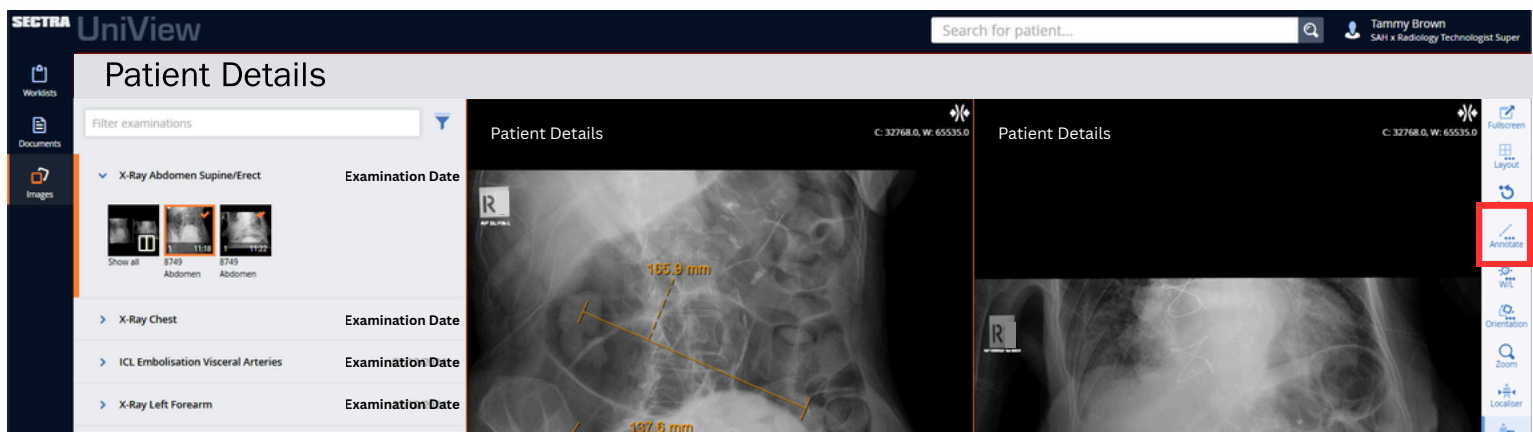

#### Quick Clinical Tips

- Always access Uniview via SanCare
- Use filters to quickly narrow exam history
- Confirm the orange border before adjusting window/level
- Use cine for rapid review of CT and MRI stacks

For Uniview help please call ext 4303 (during office hours)